

【Indications for Use】

- The Pre-Filled Normal Saline Flush Syringe is intended to be used only for the flushing of indwelling vascular access devices.

【Product Description】

- The Pre-Filled Normal Saline Flush Syringe is a three-piece, sterile, single use syringe with a 6% (luer) connector prefilled with 0.9% sodium chloride injection, USP, and sealed with a tip cap.
- The Pre-Filled Normal Saline Flush Syringe is provided with a sterile fluid path, which is sterilized via moist heat.

【Product Structure】

- It is made up of barrel, plunger, piston, nozzle cap and 0.9% sodium chloride injection(conforming with USP standards).

【Product Specification】

- 3 ml, 5 ml, 10 ml

【Sterilization Method】

- Moist heat sterilization.

【Shelf life】

- 3 years.

【Usage】

- The clinicians and nurses should follow the steps below to use the product.
- Step 1: Tear the package at the cut part and take out the pre-filled normal saline flush syringe.
- Step 2: Push the plunger upward to release the resistance between the piston and the barrel.
Note: During this step do not unscrew the nozzle cap.
- Step 3: Rotate and unscrew the nozzle cap with sterile manipulation.
- Step 4: If the product is connected to an intravenous infusion system with a needleless infusion connector, please connect the needleless infusion connector directly. If the product is connected to a traditional infusion connector that requires a steel needle connection, please connect a safety needle that meets the requirements of the OSHA bloodborne pathogen standard.
- Step 5: The pre-filled normal saline flush syringe upwardly and expel all the air.
- Step 6: Connect the product to the connector, valve, or needleless system, and flush according to the relevant principles and the recommendations of the indwelling catheter manufacturer.
- Step 7: Used pre-filled normal saline flush syringe should be disposed of in accordance with the requirements of hospitals and environmental protection departments. For single use only. No reuse.

【Contraindications】

- N/A.

【Cautions】

- Do not contain natural latex.
- Do not use if the package is opened or damaged;
- Do not use if the pre-filled normal saline flush syringe is damaged and leakage;
- Do not use if the nozzle cap is not installed correctly or apart;
- Do not use if the solution is discolored, turbid, precipitated or any form of suspended particulate matter by visual inspection;
- Do not resterilize;

- Check the expiry date of the package, do not use if it is beyond the expiry date;
- Do not place the product in a sterile interface or a sterile environment;
- For single use only. Do not reuse. Discard all the unused remaining parts;
- Do not contact the solution with incompatible medicines. Please review the compatibility literature.

【Storage】

- Product should be kept in a condition the temperature is 15-30°C, product cannot be frozen.

【Graphs and symbols used for medical device labels】

	Single use		Temperature limit
	Don't use if the package is damaged.		Keep dry
	Consult instructions for use		Date of manufacture
	Manufacturer		Use-by date
	Lot number		Do not resterilize
	Non-pyrogenic		Unique Device Identifier
	Sterilized using steam or dry heat		Keep away from sunlight
	This way up		Stacking limit by mass
	Medical device		Fragile, handle with care
	Catalogue number		Humidity limitation
	Batch code		

Edition: A/0 Date: 2023-06-18



Anhui Tiankang Medical Technology Co., Ltd.

No. 228 Weiyei Road, Economic Development Zone, Tianchang City, 239300 Anhui, China